



How To Collect a Buccal Cell Sample



PRODUCTS: A variety of Puritan swabs are suitable for buccal cell sample collection including swabs in dry transport systems.

This collection guide is for:

- DNA testing

SUGGESTED PURITAN SWABS FOR BUCCAL SAMPLE COLLECTION


Cotton	25-806 1WC
Foam	25-1805 1PF RND, 25-1506 1PF 100, 25-1605 1PF RECT
Flock	25-3206-H, 25-3306 H/U, 25-3406 H/U, 25-3706 H/U (pictured above)
Other	DNA Controlled Dry Transport Systems (FAB-SWAB) and Cap-Shure® devices.

IMPORTANT INFORMATION:


Avoid eating, drinking, and nicotine 30 minutes prior to collection.

STEP 1

Wearing gloves and other required PPE; remove the swab from the package.

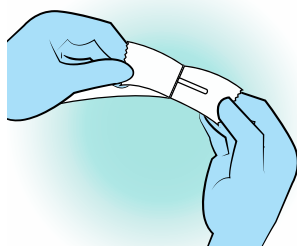


Important: Hold the swab by the handle. Do not touch the swab above the breakpoint if there is a breakpoint indicator mark on handle. Apply gentle pressure to the outside of the cheek to assure good contact with the applicator tip.

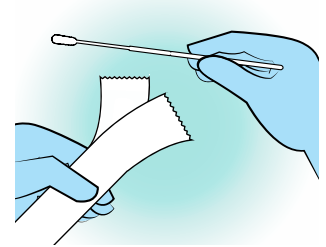


STEP 2

Insert the swab into the mouth and firmly rub up and down against the inside of the cheek 10 times.

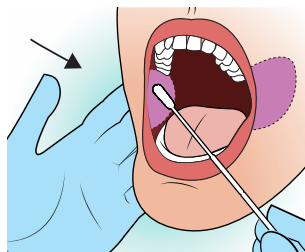

STEP 3

Repeat step 2 on other side.

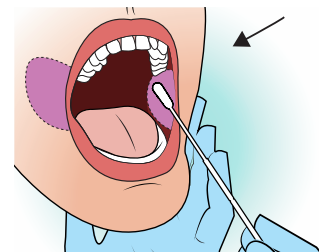

STEP 4

Remove swab from mouth being careful not to touch teeth, tongue or other surfaces.

STEP 5

After the collection allow to air dry before packaging for transport. Sample does not need to air dry if you are using a liquid processing system or an aerated transport tube device. Process immediately as per standard laboratory procedures.

This guide is for illustration purposes only. This information is not intended to be a substitute for professional, medical advice, diagnosis, or treatment. Please consult with a licensed medical professional and or the test manufacturer's product insert for specific specimen collection and transport instructions. For the most accurate and up-to-date testing and treatment guidelines, refer to the Centers for Disease Control and Prevention (CDC.gov) for more information



Puritan Medical Products Co.
31 School Street, P.O. Box 149
Guilford, Maine 04443 USA

CALL 800.321.2313 • 207.876.3311
EMAIL sales@puritanmedproducts.com
VISIT puritanmedproducts.com